

CleanSpray Faucet Filter Instructions

Disposable <0.2 µm Micro-Filters

1690	CleanSpray Faucet Filter (3 Month)
1630	CleanSpray Faucet Filter (1 Month)
48116	CleanSpray Compliance Connector + Adapter Kit



CleanSpray Faucet Filters are easy to install. The connector threads are 55/64"-27 F for connection to most standard faucets. Included is one 15/16"-27 M x 55/64"-27 M adapter and one extra flat gasket seal. (Note: Other adapters may be required for non-standard threads and are available through AquaMedix).



Read all instructions prior to installation to ensure effective & safe operation of CleanSpray devices.

Turn off water supply and disinfect faucet prior to installation.

1. Wash hands thoroughly and apply gloves to avoid contamination.
2. Remove existing aerator (end piece) from faucet if present.
3. If faucet end is male, 55/64"-27 skip to step 6.
4. If faucet end is not male 55/64"-27, an adapter will be required to convert it to male 55/64"-27 before installing the Compliance Connector.
5. Determine size of adapter required (kit QCK17 is available to help determine appropriate thread adapters).
6. Screw the Compliance Connector onto faucet (or adapters) by turning clockwise until secure. Ensure gasket seal is properly positioned (DO NOT OVER TIGHTEN).
7. Slide CleanSpray faucet filter onto Compliance Connector until it locks into place (see photo above).
8. Run hot water for 2 minutes to wet the filter before use. Inspect for water leaks around fittings. If leaks occur, remove and reattach connector (refer to step 6). If reattachment or reinstallation is required disinfect the surface of connector and filter before use.

Water flow and pressure may be slightly reduced due to the Micro-Filter. Maximum operating water pressure 75 psi.

The outlet of the filter needs to be 1" or more above the top rim of the sink. All seals must be secure. Assure no water leakage for best bacterial control. Potable water service only.

DO NOT USE Pipe joint compound or other thread sealants.

Thread seal tape (PTFE tape/plumber's tape) may be used to prevent leaks if necessary.

Filters are FDA Class I Medical Devices/EPA Pest Control Devices.

Replacing CleanSpray Faucet Filter:

AquaMedix Part #1630 (30 day) or #1690 (90 day)

Turn off water supply and disinfect tap faucet prior to installation.

1. Wash hands thoroughly and wear gloves to avoid contamination.
2. Press in white connector clip with finger or thumb.
3. Maintain pressure on white connector clip while simultaneously pulling off used faucet filter until faucet filter releases.



4. Unpackage replacement faucet filter and slide it onto the connector until it locks into place.
5. After installation, run water for 2 minutes through the filter to wet the filter before use. If leaks are present around fittings, remove and reattach the connector (refer to step 6 of installation instructions). If reattachment or reinstallation is required, disinfect the surface of connector and filter before reinstallation.

Exhausted filters can be disposed of in normal trash.

All seals must be secure. Assure no water leakage for best bacterial control.

CleanSpray Faucet Filters have been tested and meet the requirements of ASTM F838-05, for micro filtration. Filters effectively demonstrated a >99.99999% (>log 7) reduction of bacteria in the water flowing through the filters.

Operation: The Micro-Filter should be replaced when water flow decreases below acceptable levels. The Micro-Filter will continue to remove and capture contaminants in water so long as the water is flowing. Maximum filter life varies based on model (1 month or 3 months). Water systems with high concentration of particulates limit useful filter life. **A documented Micro-Filter replacement & maintenance schedule must be established after the device is installed.** CleanSpray products must be installed and used in accordance with state and local regulations.

Warning: CleanSpray products are intended for use with **potable water service only in accordance with the Safe Water Drinking Act.** These products are not intended to be used for contaminated water, untreated surface water, wastewater or water contaminated by human or animal sewage. The Micro-Filter will not remove pathogenic viruses in water. Do not attempt to clean Micro-Filter or to use damaged cartridges or Micro-Filters. Use only within product specifications and guidelines.

AQUAMEDIX, LLC

CleanSpray®

Limited Warranty

AquaMedix, LLC (“AquaMedix”) warrants that for a period of two years from the original date of purchase or up to the expiration date listed on the product or on the product packaging, this product will be free from defects in materials and workmanship.

AquaMedix, at its option, will repair or replace the product or any component of the product found to be defective during the warranty period. Replacement will be made with a new or remanufactured product or component. If the product is no longer available, replacement may be made with a similar product of equal or greater value. This is your exclusive warranty.

This warranty is valid for the original purchaser from the date of original purchase and is not transferable. Keep the original sales receipt. Proof of purchase is required to obtain warranty service. AquaMedix resellers do not have the right to alter, modify or change the terms and conditions of this warranty in any manner.

This warranty does not cover normal wear of components or damage resulting from any of the following: negligent use or misuse of the product, use contrary to the installation, use and care guide, repair or alteration by anyone other than AquaMedix or an authorized service center. Further, the warranty does not cover acts of nature, such as fire, flood, tornadoes or hurricane.

AquaMedix shall not be liable for any incidental or consequential damages caused by the breach of any express or implied warranty. Except to the extent prohibited by applicable law, any implied warranty or merchantability or fitness for a particular purpose is limited in duration to the duration of the above warranty. Some states, provinces or jurisdictions do not allow the exclusion or limitation of the incidental or consequential damages or limitations on how long an implied warranty lasts, so the above limitations or exclusions may not apply to you. This warranty gives you specific legal rights, and you may also have other rights that vary from state to state or province to province.

How to Obtain Warranty Service

Call us at (952) 479-0636 or send email to info@aquamedix.net. If it is determined that it is necessary to return the product, a **Return Authorization Number will be issued. Products returned without a Return Authorization Number will be refused.** Pack the product in the original carton, or equivalent, using newspaper or other packing materials to protect the product from damage in transit. Before sealing the carton, be sure to include a copy of the original sales receipt along with a note describing the nature of the defect or problem. **Be certain to include your return shipping address and the Return Authorization Number on the outside of the carton.**

Ship the product postage paid to:
AquaMedix, LLC
7169 Shady Oak Road
Eden Prairie, MN 55344



Upon receiving the product, we will repair or replace the product and return it to the return address on the shipping carton, postage pre-paid.

V022720-1.0